

Inspection Notes for the Sterilization Container

Please check your container before usage in terms of perceptible damages and correct operation:

1. Are the metal pieces damaged?




- No dent must be in the container box.
- The aluminium lid must not be warped.

2. Is the filter mount okay?



- The filter mount should be under prestressing and should lay rest on the complete surface.

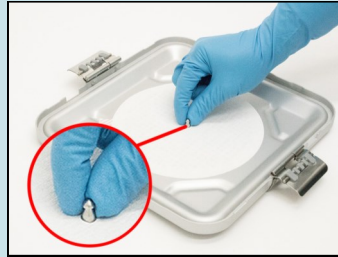


- The pin in the lid should not move.



- You should hear the filter mount click when it snaps.

3. Is the lid gasket intact?

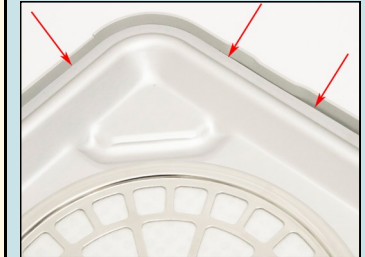

- The gasket must not have cracks, fractures etc.
- There must not be visible damages or loose components.

4. Is the filter unhurt?



- The filter needs to be unhurt: no knicks, holes, tears or slits.

5. Is the lid locker working properly and does the lid close right?



- The lock should work properly and must not be somehow damaged.



- Lubricate the lock hinges every now and then with maintenance oil.



- The lid lock has to audibly snap into the counter-part located on the box.

6. Are the handles intact?



- The handles have to be intact and must not show any damages.

7. Is the whole container accurate?









- Never use a defect container!

8. Is the seal undamaged?



- The container has to be locked with an intact seal.



	Manufacturer		Date of manufacture
	Non sterile		Batch code
	Attend to the information for use		Catalog number